NEAUVIA

RETHINKING
MEDICAL
AESTHETICS

NEAUVIA®

Fillers

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NEAUVIA®

Fillers

NEAUVIA Fillers - A comprehensive range

	INTENSE LINE							STIMULATE LINE		HYDRO DELUXE LINE	
											
	INTENSE RHEOLOGY	INTENSE FLUX	INTENSE LV	INTENSE	INTENSE LIPS	INTENSE MAN	INTENSE ROSE	STIMULATE	STIMULATE MAN	HYDRO DELUXE	HYDRO DELUXE MAN
CROSSLINKING	PEG	PEG	PEG	PEG	PEG	PEG	PEG	PEG	PEG	Non-crosslinked	Non-crosslinked
HYALURONIC ACID CONTENT	22 mg/ml	26 mg/ml	26 mg/ml	28 mg/ml	24 mg/ml	28 mg/ml	28 mg/ml	26 mg/ml	28 mg/ml	18 mg/ml	18 mg/ml
AMINO ACIDS	Glycine and L-Proline	Glycine and L-Proline	Glycine and L-Proline	Glycine and L-Proline	Glycine and L-Proline	Glycine and L-Proline	Glycine and L-Proline	Glycine and L-Proline	Glycine and L-Proline	Glycine and L-Proline	Glycine and L-Proline
CALCIUM HYDROXIAPATITE	-	-	-	-	-	-	-	CaHA	CaHA	CaHA	CaHA
INJECTION PROFILE											
SYRINGE	1 x 1 ml	1 x 1 ml	1 x 1 ml	1 x 1 ml	1 x 1 ml	1 x 1 ml	1 x 1 ml	1 x 1 ml	1 x 1 ml	2 x 2.5ml	2 x 2.5ml
NEEDLE/ CANNULA	27 G -	27 G / 30 G -	21 G 22 G	21 G 22 G	27G -	21 G 22 G	21 G 22 G	21 G 22 G	21 G 22 G	30 G -	30 G -
DURATION	6 months degradation average time ¹	6 months degradation average time ¹	6 months degradation average time ¹	6 months degradation average time ¹	6 months degradation average time ¹	6 months degradation average time ¹	The product degrades slowly after implanting ¹	6 months degradation average time ¹	6 months degradation average time ¹	Up to 4 weeks, 2 weeks average time ¹	Up to 4 weeks, 2 weeks average time ¹
AREA OF INJECTION	Face	Face	Face	Face	Lips	Face	Labia majora	Face	Face	Face and neck	Face and neck
INJECTION PLANE	Intradermal and/or subcutis	Intradermal and/or subcutis	Intradermal and/or subcutis	Subcutis and/or over the periosteum	Vermilion and/or perioral area	Subcutis and/or over the periosteum	Intradermal	Subdermis and/or over the periosteum	Subdermis and/or over the periosteum	Epidermis and dermis	Epidermis and dermis
INJECTION PLANE ILLUSTRATION											

¹ Data on file.
Intense Rheology, Intense Flux, Intense LV, Intense, Intense Lips, Intense Man, Stimulate, Stimulate Man, Hydro Deluxe and Hydro Deluxe Man are class II medical devices regulated under the EU MDR 2017/745 Regulation.
Intense Rose is a class II medical device regulated under the EU MDR 2017/745 Regulation. Manufacture: MATEX LAB SPA, via Carlo Urbani 2 ang, via Enrico Fermi, Bimbi, Italy. Please carefully read the instructions in the leaflets.
The use of these products requires the intervention of a healthcare professional. Only to be used by physicians in accordance with local legislation, trained in the injection techniques on Hyaluronic Acid based fillers.

Stimulate, has been recognized as the "Best Injectable Dermal Filler" at the AMWC 2024 Awards. This prestigious international recognition for this groundbreaking product, the only PEG-HA + CaHA filler in the market, reaffirms Neauvia's dedication to rethinking medical aesthetics.